NITRO SL

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Table of contents

Preface	9
Declaration of conformity	9
Service life	9
Disposal	9
Identification plate	9
Reuse	9
Warranty	9
Parts of the product	10
Indication	10
Scope of delivery	11
Safety instructions	11
Assembly	11
Unfolding	11
Assembly of accessories	11
Height adjustment of push handles	12
Adjusting the brakes	12
Braking Parking	12
Adjusting the backrest	12
Sitting	12
Folding Transporting Stowing	13
Cleaning Maintenance	13
Technical data	13

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

The manufacturer declares conformity for the product described in accordance with the regulation of council directive 93/42/EEC dated June 14, 1993 on medical devices and directive 2007/47/EC.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

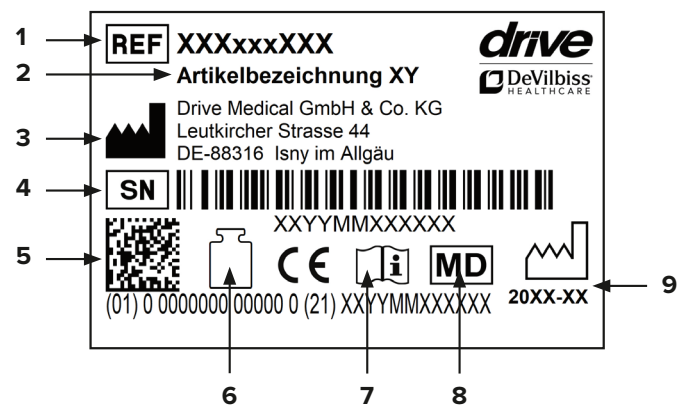1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes and handles

Parts of the product

NITRO SL

- | | | |
|-----------------------------------|-------------------|------------------|
| 1. Handle | 5. Seat | 9. Pocket |
| 2. Push handle | 6. Rear wheel | 10. Back support |
| 3. Brake | 7. Steering wheel | 11. Bowden cable |
| 4. Locking lever for push handles | 8. Cane holder | |

Indication

INDICATION:

Walking aid for outdoor and indoor use to compensate for or improve limited walking ability. Insured parties are to be provided with walking aids with the intention of compensating for or at least improving the user's limited walking ability (expansion of radius of activity or mobility). Persons with limitations of hand/arm functions require particular adjustments/designs of handles or handrests/

armrests. Movable walking aids may not be suitable for persons with balance disorders; in individual cases, this should be tested out. Where appropriate, an attendant must be present. The rollator is to be used only for normal, everyday activities.

CONTRAINDICATION:

The rollator is not suitable for use in the case of: perception disorders, severe loss of balance, inability to sit down.

Scope of delivery

1 x	Rollator
1 x	Pocket
1 x	Back support
1 x	Cane holder
1 x	Instructions for use

Safety instructions

- Use the rollator only as a walking aid!
- Use the rollator only on an even and solid surface.
- Rollator may not be used as a seat in vehicles, such as a bus or train!
- It is prohibited to stand on the rollator or to transport a person or animal on the rollator.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.
- Use the tray only for small, lightweight objects which are not sharp or hot. Max. load: 5.0 kg
- Use the basket only to transport objects and do not overload it! Max. load: 5.0 kg
- Use on escalators and the like is expressly prohibited!
- Do not hang any bags on the handle – risk of tipping!
- On steep surfaces, use the service brake (pull the brake lever upwards).
- Leaning excessively on the handles can cause the rollator to tip backwards.
- When folding/unfolding the rollator, clothing and body parts can be pinched.
- Protect the rollator from direct sunlight. Product parts can become hot and cause skin injuries if touched.
- Prior to each use, ensure that your rollator is intact and all adjustable parts are properly secured. Check that the brakes are working properly!
- Before using the rollator, ensure that the folding guard is properly locked.
- Check the screws on the rollator regularly and tighten, if necessary.
- The braking effect can be impaired by wear and tear on the tires. For this reason, check the braking effect and tires regularly.
- The rollator should NOT be carried by the seat, removable parts or the brake cables.
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and

the competent authority of the member state in which the user and/or patient is located.

Assembly

- Take all the parts out of the box.
- Unfold the frame.
- Insert the castors, making sure that you position the forks pointing outwards when the rollator is in motion.
- “R” label = right side
- “L” label = left side

Caution: Make sure that the two push-buttons are fully and properly engaged.

Straighten the frame and insert the rear wheels. Guide the rear wheels inside through the folding mechanism so that the bowden cables do not run outside of the frame. If the brake pad is rubbing on the wheel, take the assembly back off and unwind the cable a few turns. Adjust the cable using the screw situated at the base and/or on the handle side.

Unfolding

Place the rollator on its wheels. Now press both side rails of the seat section downward until they click into place (**pic. 1**).

The rollator is unfolded correctly when both locking hooks have clicked into place (**pic. 2**).

If it is difficult to engage the seat section while unfolding, this can be remedied by applying a small film of grease to the locking hooks.

Assembly of accessories

The pocket is hung by its loops from the ends of the frame tubing.

Height adjustment of push handles


Insert push handles at least to the first cross hole into the frame tubing. Insert the fixing screw from the inside and tighten using the washer and star screw (pic. 4).

To change the height adjustment of the push handles, loosen the fixing screw by turning the star screw counter-clockwise. Then lock the handle at the desired height, as described above (pic. 5).

Adjusting the brakes

If braking is poor or too hard, the brake can be adjusted either at the brake housing above (pic. 6) or using the adjusting screw on the Bowden cable below (pic. 7) to achieve the desired braking effect.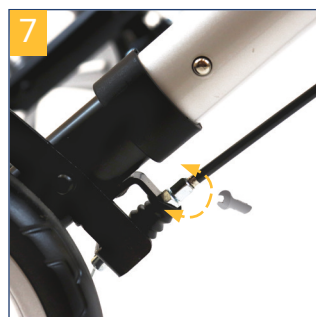

The brake is correctly adjusted when the brake lever as well as the brake wires are free of play and the wheels can be rotated without grinding noises when the brake is released.

Braking | Parking

To brake when walking with the rollator, pull both brake handles upwards with your hands (pic. 8). Always brake slowly while applying equal pressure with both hands.

To apply the brakes/park the rollator, press both brake handles downwards until they audibly click into place (pic. 9)! To release the brake, pull both brake handles upwards once again until they are released (brake handles are in the middle position).

The braking effect can be impaired by wear and tear on the tires. For this reason, regularly check the braking effect.


Adjusting the backrest

Position the sling by inserting the fixings on each side into the guides on the brackets situated at the level of the handles. Adjust the supports to the height required.

Caution: Before each use, make sure that the sling has been inserted correctly and that the two brackets are fully tightened. Make sure that you fit the sling the right way (stitching facing outwards).

Sitting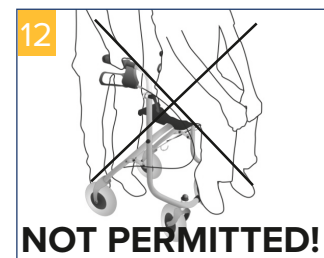

Place the rollator on an even and solid surface. Then apply the brakes. The rollator must be stable and may not slip. Only sit upright; do not lean backwards (pic. 11)!


Seated position permitted only as shown and against the direction of movement (pic. 12)!

Cane holder

Clip the upper cane holder below the yellow reflector to the frame. The lower cane holder part must be fixed on the lower frame part. Make sure correct assembly. The cane holder can be mounted left or right.

Folding | Transporting | Stowing

To fold up the rollator, use the handle to pull the seat section upwards. (pic. 13-14).

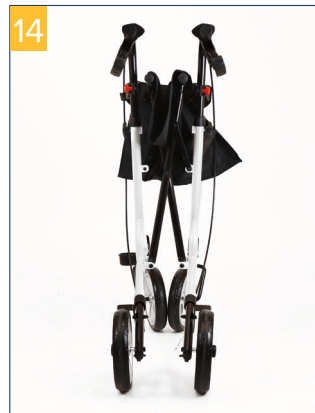Remove accessories, if necessary!

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60° C for long periods of time (for example, use in a sauna, etc.)!
- In the event of defects, contact your specialist dealer from whom you purchased this product!
- Check the condition of the tires regularly. They are “fail-safe”, meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage, however.

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Drive Medical GmbH & Co. KG
Obelis S.A
Ralon Medical Equipment CO.Ltd