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Vida 13" und 22"

DE

EN

Enjoy mobility.

Inhalt

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6.	Use	22
6.1.	Entering and exiting from the side	22
6.2.	Entering and exiting from the front	23
6.3.	Climbing over stairs or a high step	24
6.4.	Drive mode	24
7.	Technical data	27
8.	Care instructions for the user	30
8.1.	Cleaning	30
8.2.	Disinfection	31
8.3.	Checking the wheelchair by the user	32
8.3.1.	Brake check	32
8.3.2.	Checking the tyres	32
8.3.3.	Maintenance note	33
8.4.	Maintenance advice for the	33
8.5.	Specialist trade	33
9.	Repairs	34
10.	Storage / Shipping	34
11.	Passing on the wheelchair	34
12.	Disposal	35
13.	Transport	35
14.	Warranty	36

1. Content

Dear user,
you have chosen a wheelchair from Bischoff & Bischoff. We thank you for your trust.

This instruction manual contains all the necessary information for operating the Vida wheelchair.

Your wheelchair is designed as a transport wheelchair for use indoors and, to a limited extent, for outdoor use.

Please read and observe the operating instruction carefully before using your new wheelchair for the first time. This is an integral and necessary part of the wheelchair. Keep the instruction manual handy and give it to the person who will receive the wheelchair.

For users with visual impairments, this document is accessible as a PDF file from our website: www.bischoff-bischoff.com

Repair and adjustment work requires special technical training and may therefore only be carried out by specialist dealers authorised by Bischoff & Bischoff GmbH.

Notice!

Despite careful research and editing, incorrect information may have crept into our operating instructions. We reserve the right to make technical changes. Illustrations may contain accessories. The latest version of the operating instructions can be found in the download area on our website:

www.bischoff-bischoff.com

1.1. Signs and Symbols

Important! Indicates particularly useful information in the respective subject context.

Attention! Indicates particularly safety-relevant information. Observe operating instructions!

2. Important safety instructions

2.1. General safety instructions

To avoid falls and dangerous situations, you should first practise using your new wheelchair on level, manageable terrain or in an obstacle-free area. An accompanying person is recommended in this case.

- Only drive on level, solid ground.
- Be aware of the risk of entrapment between moving parts when adjusting, using and maintaining the wheelchair.
- The accompanying person should be physically and mentally capable of driving a wheelchair.
- The driver and the accompanying person must not be under the influence of alcohol or drugs.
- Before using the wheelchair, please check that all attached parts are properly fastened.
- Lift the wheelchair only by firmly mounted parts (the armrests or the frame parts).
- Make sure that the quick-release axles are correctly seated in the axle sockets, otherwise there is a risk of injury.
- Keep clothing and body parts away from gaps between moving parts!

- Please note that the risk of tipping over increases when there is a shift in balance (e.g. due to strong upper body movements or when overcoming obstacles).
- When negotiating obstacles (steps etc.), an attendant must be called in. Use the tipping aid (see chapter 6.4) to tilt the wheelchair accordingly.
- Stairs and larger obstacles may only be overcome with the help of at least two accompanying persons. Use the lifting points for this (see chapter 6.3). If facilities such as ramps, lifts or stair-climbing aids are available, use them (Fig. A).
- When driving on a slope/gradient, it must be noted that depending on the set axle position, there is a risk of tipping over from just a few degrees.
- Avoid collisions of any kind! Avoid driving into an obstacle (step, kerb) without braking or jumping down from a step (Fig. B).
- Replace the tyres as soon as the tread is worn down.
- When getting in or out of the wheelchair, apply both parking brakes.
- Use the parking brake only for parking, it must not be used to slow down the drive.
- Avoid driving on uneven, loose surfaces.
- The Road Traffic Act (StVZO) must be observed in road traffic (see chapters 2.3 and 6.4).
- The seat and upholstery materials meet the requirements of resistance to flammability according to EN 1021- 2, nevertheless do not expose upholstery or other parts to fire sources - such as cigarettes.
- The wheelchair may only be used to transport one person. The wheelchair must not be used to transport loads or more than one person.
- When braking from high speed or on long downhill stretches, using the hand rims, fingers and palms heat up. Caution, risk of burns! (Fig. C).
- Avoid fast journeys and downhill stretches.
- Note that upholstered parts that are exposed to direct sunlight can heat up and cause skin injuries when touched. Therefore, cover these parts or protect the wheelchair from sunlight.


Protect the wheelchair from sunlight to avoid burns from heated components.

- Also note that the frame and upholstered parts can cool down considerably in winter. If possible, do not park the wheelchair outside in cold weather.
- Observe the storage and operating conditions.
- When transferring between wheelchair and bed, please always keep in mind:
 - apply both parking brakes.
 - swing the side part of the entry/exit side downwards.
 - swing the foot plates up or away (when getting in/out).
- Reporting of incidents:

If you, as the operator, user or their relatives, discover possible defects or functional limitations, please contact an authorised specialist dealer immediately. Operators, users or their relatives should inform their specialist dealer who provided the product of any suspected serious incidents which have had or could have one of the following consequences, either directly or indirectly, and which affect them. You can also report this to the competent higher federal authority.

 - the death of a patient, user or other person,
 - the temporary or permanent serious deterioration of the state of health of a patient, user or other person,
 - a serious threat to public health.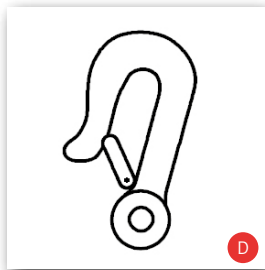

2.2. Safety in vehicles

You can tell whether or not your wheelchair is approved as a seat in a motor vehicle for transporting mobility-impaired persons by the symbol on the type plate on the wheelchair:

- Wheelchair suitable as a seat in a motor vehicle for transporting persons with reduced mobility (Fig. D).
- Wheelchair not suitable as a seat in a motor vehicle for transporting mobility impaired persons (Fig. E).

The Vida wheelchair is not suitable as a vehicle seat, therefore it must not be used as such under any circumstances. It does not meet the requirements of ISO 7176-19. Failure to do so may result in serious injury and death in the event of an accident.

The Vida wheelchair is not suitable as a vehicle seat. In the event of an accident, this can lead to severe injuries and death.

2.3. Participation in road traffic

The wheelchair is designed for indoor use and, to a limited extent, for outdoor use. Please note that if you participate in public road traffic, you must comply with the road traffic regulations. Please observe chapter 6.4, Driving.

Do not endanger other participants by driving recklessly, especially on pavements.

2.4. Brakes

Service brake:

Vida with 13" wheels:

In driving mode, the wheelchair is braked by the attendant via the two brake levers on the push handles.

Vida with 22" wheels:

When driving, brake the wheelchair using the handrims. Please note that your hands may get hot.

Parking brake/parking:

Vida with 13" wheels:

As soon as the wheelchair is stationary, pull the hand levers until the catches on the front part of the brake levers can be engaged by pushing them forwards.

Vida with 22" wheels:

As soon as the wheelchair is stationary, push both brake levers forward. (see chapter 4.5). The wheelchair is safely braked when it can no longer be pushed away and both brake levers are pushed forward as far as they will go. Pull the lever back towards you to release the brake.

2.5. Purpose

The wheelchair is designed exclusively for increasing the mobility and transport of adults with walking disabilities with the indicated indications. Unless otherwise described, the instructions in this user manual are directed to the occupant as the operator.

The maximum user weight is 100 kg.

Please note:

We can only provide a warranty if the product is used under the specified conditions and for the intended purposes.

2.6. Indications

Inability to walk or severe walking disability in adults due to:

- Paralysis
- Limb loss
- Limb defect/deformation
- Joint contractures/joint damage (not on both arms)
- Musculoskeletal/neuromusculoskeletal movement disorders)
- Limited coordination of the arms or hands
- Other diseases
- Use as a transport wheelchair for short-term indoor and outdoor
- Use with an assistant or caregiver.

2.7. Contraindications

The use of the wheelchair is unsuitable for:

- • Perceptual disorders
- Severe balance disorders
- Loss of limbs on both arms
- Joint contractures/joint damage in both arms
- Incapacity to sit
- Reduced or insufficient vision
- Severe limitations of cognitive abilities
- Outdoor use is limited, the small rear wheels make it difficult to negotiate obstacles such as kerbs.

2.8. Declaration of conformity

Bischoff & Bischoff GmbH, as manufacturer, declares under its sole responsibility that the wheelchair Vida complies with all requirements of MDR 2017/745 that are applicable.

2.9. Responsibility

We shall only provide a warranty if:

- the product is used under the specified conditions and for the intended purpose,
- modifications, extensions, repairs and maintenance work are only carried out by persons who have been authorised by us to do so, and the wheelchair is used in compliance with all instructions for use.

2.10. Useful life

The expected service life is up to 5 years if the wheelchair is used as intended and the safety, care and maintenance instructions are followed. The wheelchair can be used beyond this period if it is in a safe condition.

3. Product and delivery overview

3.1. Delivery inspection

All Bischoff & Bischoff GmbH products are subjected to a proper final inspection in our company and are certified with the CE mark. The wheelchair is delivered in a special box in factory settings. After unpacking, keep the carton if possible. It is used for possible later storage or return of the product.

Delivery and instruction are usually carried out by qualified medical supply retailers.

In the case of shipment by rail or forwarding agent, the goods must be checked immediately for transport damage in the presence of the deliverer.

Check the contents for completeness and integrity. In case of irregularities and damage, contact our customer service.

3.2. Scope of delivery


After receiving the goods, please immediately check the contents for completeness.

The contents consist of:

- Outer packaging
- Wheelchair preassembled
- Operating instructions

3.3. Type plate and serial number

The type plate (Fig. 01) and the serial number are located at the bottom of the frame.

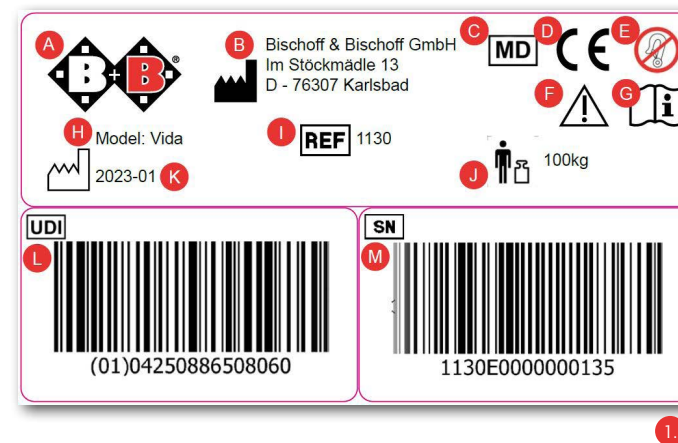- A Manufacturer's sign
- B Manufacturer's address
- C Medical product
- D CE mark
- E Observe the fastening approval in the vehicle
- F Important! Observe operating instructions
- G Observe instructions for use
- H Model
- I Model number
- J Max. user weight
- K Production date
- L UDI number
- M Serial number

3.4. Product overview

3.4.1. Product overview Vida 13"

3.4.2. Product overview Vida 22"

4. Assembly / Adjustments

You can adjust the settings of your wheelchair yourself with a little manual skill. If you have any questions, please contact your specialist dealer.

Watch your fingers when making adjustments to avoid the risk of pinching.

4.1. Fold and unfold

Delivery




Your new Bischoff & Bischoff wheelchair is delivered completely assembled and folded in a Bischoff & Bischoff original cardboard box (Fig. 4).

To avoid damage during transport, plug-in attachments are supplied separately (Fig. 6).

Unfold

Position your folding wheelchair next to you. Tilt it towards you so that one drive wheel is unloaded. Now push the seat tubes apart and push them down with the flat of your hand until they are locked in the holder on the frame (Fig. 5).

Insert the leg supports and then swivel them inwards until they lock into place (Fig. 6). Fold the footrests down and attach the calf strap (Fig. 7).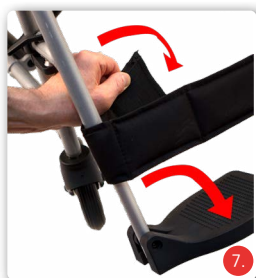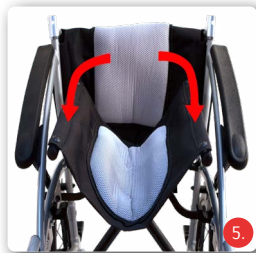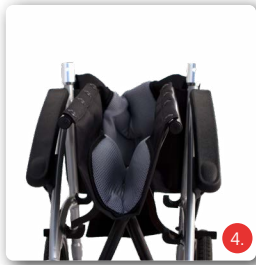

Please take care of your fingers when unfolding. Danger of pinching!

Now fold the back section upwards until the push handles audibly engage (Fig. 8). Secure the wheelchair against rolling away as follows:

- On the Vida 13" lock the brake levers, see chapter 4.5.
- On the Vida 22", operate the knee lever brake by pushing the lever forward, see chapter 4.4.

Fold the armrests upwards (Fig. 9)

Folding


Remove the calf strap from your wheelchair and fold the footrests upwards. Pull the lever on the upper part of the leg support upwards and fold the leg supports outwards and then pull them away upwards. (Fig. 10).

Next, fold down the push handles by pressing down the levers located below the brake levers on the back tube until the push handles can be folded down.

Now pull the two seat tubes upwards on the outer sides of the seat. If necessary, you can also fold the armrests down. To do this, press the button located on the lower part of the armrest guide strut and push the armrest downwards (Fig. 12).

Please take care of your fingers when folding. Danger of pinching!

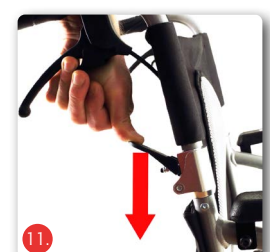4.2. Legrest

The Vida is equipped with removable leg supports (Fig. 6). The two leg supports are delivered assembled with the foot plates. The foot plates on both leg supports can be folded upwards in the sitting position.

Fitting the legrest

Hold the legrest by the upper part and guide it into the legrest holder so that it points outwards. Swivel the legrest forward until it locks into place (Fig. 13). Fasten the calf strap with the Velcro fasteners located at the ends (Fig. 7).

 Check that the legrests are firmly seated before starting your journey!

Swinging down the legrest

To make getting in/out of your Bischoff & Bischoff folding wheelchair as easy as possible, you can simply swing the legrests outwards or inwards if necessary without having to remove them.

- Pull the lever. This opens the lock (Fig. 14).
- Now you can swing the legrest outwards or inwards (Fig. 15).
- If necessary, fold up the foot plates beforehand.


To restore the original state, simply swing the legrest forward until it locks into place.

 When standing up from the wheelchair, never step on the footplates!

Adjusting the legrest length

The length of the leg support can be continuously adjusted by a total of 12 cm as standard. To do this, loosen the Allen screw (hexagon socket 5 mm) and adjust the legrests according to your choice (Fig. 16).

 The maximum extension length is indicated with a  Make sure that both legrests are set at the same height and that the adjustment screws are tightened!

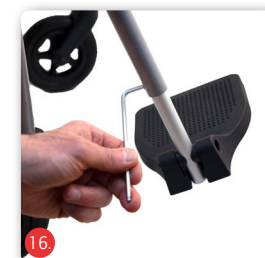4.3. Height adjustable armrests

Swivel armrest

To make it easier to get in/out of the chair, the two armrests can be swivelled downwards.

Press the locking button with light finger pressure and swivel the armrest downwards (Fig. 17).

Height adjustment:

- Press the push button (Fig. 18) to adjust the armrest to the desired position.
- When you have reached the desired position, release the push button. The lock will then engage.
- The height is correct when the upper arm and forearm form approximately a right angle with the forearm lying on top.

4.4. Brakes

Operating the knee lever parking brake (compact brake)

To lock the brake on the Vida 22", simply push the brake lever forward until it engages (Fig. 19). To release the parking brake, simply pull the brake lever backwards.

Adjusting the knee lever parking brake (compact brake)

If you notice that the braking effect is insufficient or uneven, proceed as follows:

- Check the tyre tread depth. Also check whether there is even wear. If the tread depth in the centre of the track is < 1 mm, the tyre must be changed by a specialist dealer.
- Now check the adjustment of the brake. It must be adjusted so that the distance between the wheel tread and the brake bolt is 12 mm when the brake is open. The brake pin must be horizontal, i.e. parallel to the ground.
- After loosening the Allen screw (5 mm) on the brake holder (Fig. 20), you can position the brake correctly.
- The compact brakes only act on one drive wheel at a time!
- After completing the adjustment process, tighten the loosened screw again and carry out a brake test!
- Adjustments to the brake system should only be carried out by an authorised dealer!

4.5. Parking brake for the attendant

The Vida 13" wheelchair is equipped with a parking brake for the attendant. To lock it, pull the brake lever and push the small locking lever down (Fig. 21). To release the lock again, pull the brake lever slightly and the locking lever will automatically spring back to its original position.

Generally!

Adjustments to the brake system should only be carried out by an authorised dealer!

4.6. Seat height

The seat height cannot be adjusted on the Vida.

4.7. Seat angle

The seat angle cannot be adjusted on the Vida.

4.8. Wheels and Tyres

The spoked wheels are firmly screwed to the chassis with a machine screw (M15) (Fig. 22).

Drive wheels

PU tyres (puncture-proof, without valve) of size „13" or „22" are fitted to your Vida as standard. In case of wear or defect, the authorised specialist dealer must be commissioned with the repair.


Steering wheel

Puncture-proof tyres (without valve) of size 150 mm are mounted on the Vida as standard (Fig. 23). In case of wear or defect, the authorised specialist dealer must be commissioned with the repair.

5. Accessories

5.1. Calf strap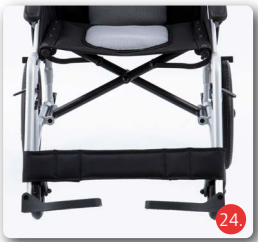

For better leg support, a calf strap is available as a standard accessory for the Vida (Fig. 24).

6. Use

6.1. Entering and exiting from the side

- Bring the wheelchair and the current or future seating surface as close to each other as possible.
- If you want to transfer from another wheelchair or indoor undercarriage, secure it by applying the parking brakes.
- Secure the Vida against unintentional rolling away by operating the parking brake or hand lever brake.
- Lower the armrest down
- Fold the footplates / footrest to the side or remove the leg supports.
- Now slide sideways onto the other seat.
- Make sure you sit as far back in the seat as possible.
- Finally, hang the legrests back in the starting position. And swing the armrests up again if necessary.

6.2. Entering and exiting from the front

Before getting on/off the wheelchair, the parking brake or hand lever brake must be applied. Trips outside the home should only be made with sufficient driving experience and with an assistant or carer.

- Swivel the foot plates upwards or remove the leg supports. Bring the Vida and the current or future seating unit as close to each other as possible.
- Secure the Vida against unintentional rolling away by operating the parking brake or hand lever brake.
- If you want to transfer from another wheelchair or indoor undercarriage, secure it by applying the parking brakes.
- Now slide onto the seat by turning your body.
- Finally, hang the leg supports back in the starting position and swing the foot plates down.

Never climb onto the foot plates! Danger of tipping!

6.3. Climbing over stairs or a high step

- The helpers are only allowed to grab firmly mounted frame parts.
- The lifting points are shown in figure 25.
- When going up a staircase, the rear helper pulls the wheelchair over the step using the permanently mounted handles.
- The lower helper grips the front frame tubes and stabilises the position. In doing so, he presses the rear wheels of the wheelchair against the steps.
- When descending, the lower helper brakes by pushing the wheelchair straight against the steps. The upper helper holds the wheelchair by the push handles, secures it and keeps it in the correct position.
- The wheelchair should roll down one step at a time.

The holding points for transporting the wheelchair are exclusively the frame and the push handles. Armrests must never be used to lift the wheelchair!

For obstacles and stairs with more than three steps, two attendants must help.

6.4. Drive mode

Sit in your Bischoff & Bischoff wheelchair. Please make sure that you sit as far back as possible. The locomotion and braking is done by the handrims on the drive wheels (Vida 22"). With the Vida 13", locomotion and braking are carried out by the accompanying person. Carry out the first driving attempts carefully until you have become accustomed to the wheelchair and its driving behaviour.

Wheelchairs have only limited tipping and sliding safety. Special care should be taken when braking, starting or turning, on slopes or inclines. Extreme leaning out of the wheelchair also increases the risk of tipping over. For greater safety, all wheelchairs can be equipped with a lap belt (not standard).

You can hang shopping bags or a backpack on the frame of the backrest.

In order to be able to overcome obstacles more easily, the attendant must position the wheelchair at the correct angle to the obstacle by pressing on the tipping aids with the foot (Fig. 26).

An accompanying person may only push the wheelchair by the push handles (Fig. 3 and 4), not by the backrest, as there is a risk of injury.

Note that the risk of tipping backwards increases significantly when bags and backpacks are attached.

According to German StVZO 66/1 of 01.01.1981, hand-operated wheelchairs that participate in public road traffic (including on pavements) require fixed lighting

This consists of (passive):

- two red reflectors to the rear
- two yellow reflectors on the sides (on both sides).

If the wheelchair is used after dark, the following are additionally required (active):

- one white light towards the front
- a red light to the rear.

The functioning of these lights must be ensured by a circuit (batteries) independent of driving operation.

The Vida wheelchair has no active or passive lighting.

Before driving off, if appropriate, check that the lighting system is in working order.

Please take care of your fingers when using it. Danger of pinching!

7. Technical data

Dimensions and weights

	Vida 13"	Vida 22"
User weight (kg)	100	100
Seat width (mm):	450	450
Total length with leg supports (mm):	990	1030
Width (mm):	645	680
Height (mm):	890	890
Folded (BxHxL) (mm):	280 x 720 x 785	340 x 720 x 790
without / (with legrest)	(975)	(1020)
Lower leg length (mm):	400 - 480	410 - 490
Handle height (mm):	860	880
Seat depth (mm):	390	390
Seat height front (mm):	480	480
Seat height rear (mm):	460	460
Backrest angle (°):	74	75
Seat angle (°)	3,7	3,5
Height of backrest cushion (mm):	430	430
Height of armrest (mm):	100 - 235	100 - 235
Length of armrest (mm):	360	360
Width of armrest (mm):	50	50
Front attachment of the armrest (mm):	250	250
Handrims Ø (mm):	not applicable	495
Position of rear axle with regard to the backrest (mm):	0	0

	Vida 13"	Vida 22"
Weights		
Weight without attachments (kg):	10,2	11,4
Weight attachments - 2 legrests (kg):	1,2	1,2
Driving characteristics		
Steering radius (mm):	525 - 553	555 - 570
Ground clearance (mm):	40 - 110	35 - 105
Steering range (mm):	1050 - 1105	1110 - 1140
Stability		
Parking brake function up to Inclination (°).	7	7
Environmental conditions		
Temperature (°C):	-20 - 40	-20 - 40
Humidity (%):	max. 80	max. 80
Storage conditions		
Temperature (°C):	-25 - 40	-25 - 40
Humidity (%):	max. 60	max. 60
Tyres		
Drive wheels (Zoll)	13"	22" x 1 3/8
Steering wheels (mm):	150	150
Material of all wheels	Rubber	Rubber
	Vida 13"	Vida 22"

Equipment		
Side panels:	Fold down	Fold down
Legrest:	Height adjustable	Height adjustable
Brakes:	Handbrake for accompanying person	Knee lever brake
Axles drive wheels:	Screw axle M15	
Frame:	Steel, powder-coated to protect against corrosion	
Attachments:	Legrests with footplate, calf strap.	
Seat and back upholstery:	Foam with nylon cover, flame retardant	
Armrest:	PU foam	
Accessories:	Calf strap (standard equipment)	

Dimensions given are subject to manufacturing tolerances of +/- 10mm.

8. Care instructions for the user

8.1. Cleaning

The seat and back materials can be cleaned with warm water using a sponge and/or a soft brush. In case of stubborn dirt, a commercial cleaning agent can be added to the water.

Note: Never use aggressive cleaning agents such as solvents / scouring agents or hard brushes.

Plastic parts are best cleaned with a commercially available plastic cleaner. Please observe the product information of the cleaning agent.

- Plastic claddings are attacked by non-ionic surfactants and solvents, especially alcohols!
- Wipe the frame parts of the wheelchair with a damp cloth. For heavier soiling, also use a mild cleaning agent.
- Clean the wheels with a damp brush with plastic bristles (do not use a wire brush!).
- Do not use a high-pressure cleaner.
- Wash the push handles, arm pads, seat and back pads with a mild detergent.
- Upholstery parts can be machine washed at 40°C. Do not machine dry, only dry at room temperature.

8.2. Disinfection

- Before disinfection, the wheelchair must be thoroughly cleaned. The wheelchair can be disinfected with a household disinfectant. Use approved spray disinfectants or alcohol-based wipe-on disinfectants.
- In the case of multiple product users, a commercial disinfectant is prescribed.
- Use only wipe-on disinfectants approved by the Robert Koch Institute (RKI) and the Association for Applied Hygiene (Verbund für Angewandte Hygiene e.V.) (e.g. Bacillol AF or Kohrsolin FF); see manufacturer's recommendations for exposure time and concentration.
- Caution: Damaged upholstery parts cannot be sufficiently disinfected. These should be replaced.
- Under certain circumstances, the surfaces can be affected by disinfectants. Longer-term restrictions of the functionality can be the result. Please note our enclosed overview of the most important care, hygiene and maintenance tasks.

Attention: Danger of infection! If there are several users, the wheelchair must be cleaned after each use.

Attention! Do not use a high-pressure or steam cleaner

Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your specialist dealer will be happy to provide you with these.

Please note our listed overview of the most important care, hygiene and maintenance work.

8.3. Checking the wheelchair by the user

8.3.1. Brake check

Before each journey, check that the brake system is functioning properly.

When the brakes are applied, it should not be possible to push the wheelchair away. When the brakes are released, the wheelchair should be easy to drive without grinding noises and with good directional stability.

If you notice any unusual braking behaviour, inform your specialist dealer immediately and remove the wheelchair from circulation.

8.3.2. Checking the tyres

Before each journey, check the tyres for integrity. If the tyre is damaged, have it repaired by an authorised specialist workshop.

8.3.3. Maintenance note

We recommend that the wheelchair is inspected by a specialist dealer at least once a year.

In the event of faults or defects on the wheelchair, it must be handed over immediately to the specialist dealer for repair.

The repair and replacement of parts is carried out by the specialist dealer.

The necessary information and documents for repair and maintenance will be made available to the specialist dealer on request.

If you have any questions, please contact the Bischoff & Bischoff customer service

You must check your wheelchair regularly according to the maintenance schedule and find it to be in perfect working order.

8.4. Maintenance advice for the specialist trade

Maintenance must be carried out in accordance with the Bischoff & Bischoff maintenance plan. A maintenance schedule will be provided upon request. A training programme for the specialised trade is also available.

9. Repairs

Repairs and maintenance measures may only be carried out by authorised partners (medical supply stores or specialist dealers). To find a partner near you, please call our customer service

10. Storage / Shipping

If the wheelchair is to be stored or shipped, all insertable and unattached parts, except drive wheels, must be removed and packed individually in suitable cardboard boxes. The individually packed parts can then be packed together in a larger box. It is recommended to keep the original packaging and store it for these purposes so that it is available when needed. Then your wheelchair is optimally protected against environmental influences during storage or transport. For the storage of your wheelchair, please note the specifications in the technical data. To avoid mould and damage to the upholstery parts, the storage location should be as dry as possible and not be exposed to direct sunlight.

11. Passing on the wheelchair

The wheelchair is suitable for reuse. Before it is passed on to another user, it must be serviced and hygienically prepared by a specialist dealer. When passing on the wheelchair, please remember to give this user manual and the address of your specialist dealer to the new user.

 The wheelchair must be checked according to the maintenance schedule and be in perfect working order.

12. Disposal

If your wheelchair is no longer in use and is to be disposed of, please contact your specialist dealer

 If you want to dispose of the waste yourself, ask local recycling companies about the regulations in your area.


13. Transport

You can tell that your wheelchair is not approved as a seat in motor vehicles for the transport of mobility-impaired persons (MIP) by the symbol on the type plate (Fig. 27):

 The wheelchair is not crash tested and must therefore never be used as a seat in a MIP.

Your wheelchair should only be transported when folded (Fig. 28). When folded, the wheelchair should be secured to fixed frame parts in the vehicle.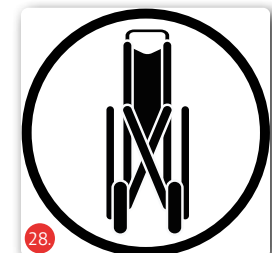

14. Warranty

1. For products supplied by Bischoff & Bischoff, the period of warranty is 24 months from purchase date. The products are delivered free from production and material defects. In case of evidence of material or manufacturing defect, defective parts will be replaced free of charge.
2. This warranty excludes claims, if a repair or replacement of a product or a part is required for the following reasons:
 - a) normal wear and tear, which include but is not limited to the following parts where fitted; Batteries, motor brushes, handles, armrests, upholstery, tyres, brakes, caps, etc.
 - b) overloading the product such as, for example, by exceeding the maximum user weight or load.
 - c) the product or part has not been maintained or serviced in accordance with the Instruction manual, the care and hygiene instructions or the manufacturer's recommendations as shown in the maintenance instructions.
 - d) Accessories have been used which are not original accessories.
 - e) The product or part has been damaged through negligence, accident or incorrect use.
 - f) Changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.
 - g) Repairs have been carried out, before our customer service has been informed of the circumstances.
3. To enforce the warranty, please contact your dealer with the exact details of the nature of the difficulty. The product must be repaired by a dealer authorised by Bischoff & Bischoff.
4. For parts, which have been repaired or exchanged within the scope of the warranty, the warranty will be extended to the remaining warranty period for the product in accordance with point 1.
5. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, following the fitting.
6. If the repair fails after a reasonable time, the customer can, at his discretion, request a reduction, compensation or the termination of the purchase contract.

7. The guarantee is subject to the law of the country in which the product was purchased from Bischoff & Bischoff. Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your dealer will be happy to provide these to you. For replacement devices and repairs, the warranty period is one year. Observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your dealer will provide these to you.

Non-observance of the operating instructions as well as improper use, maintenance jobs and technical changes or adaptations without the permission of the Bischoff & Bischoff company, lead to guarantee loss and the loss of product liability in general.

Warranty certificate

Product: Vida

Model number:
Serial number: *
* (to be filled in by the dealer)

Dealer:

Date & Stamp
(Pay attention to our general terms and conditions)

*Signs are located on the seat bracket

Note!

Print errors, product and colour changes and technical information are excepted. The colours of the products shown may be slightly different in real life. The images may show accessories. The current version of each Instruction manual is ready for download on our website www.bischoff-bischoff.com.

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