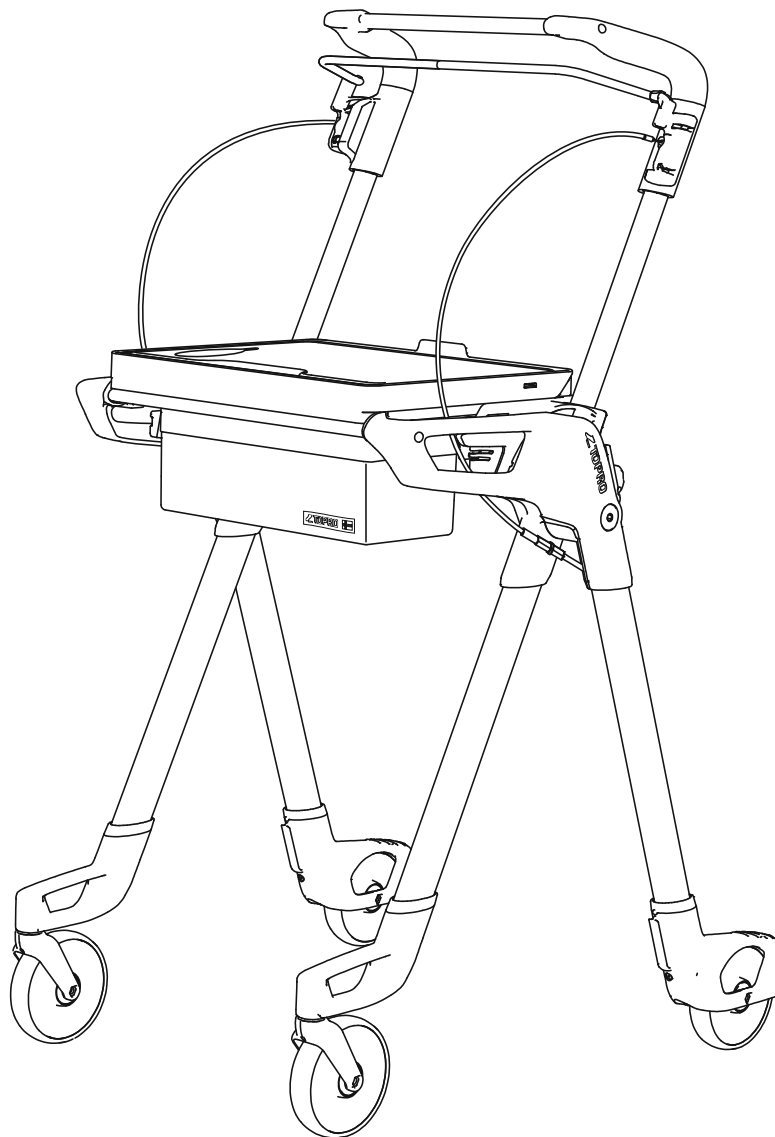
TOPRO

Instructions for use TOPRO Hestia

topromobility.com

made in Norway

EN

Illustrations
Measurements
Technical data
Article numbers

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
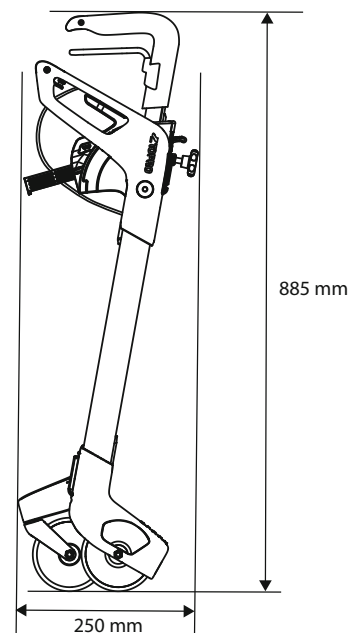
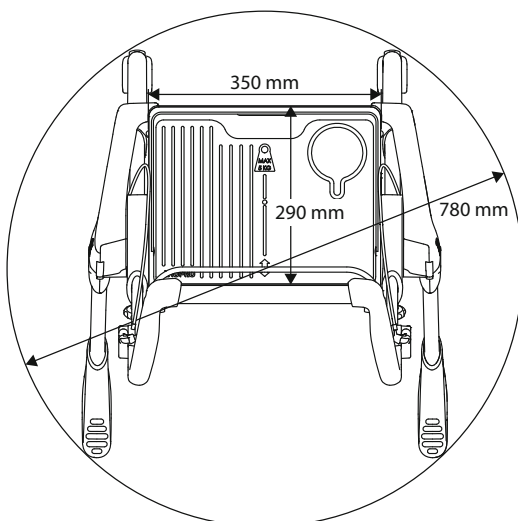
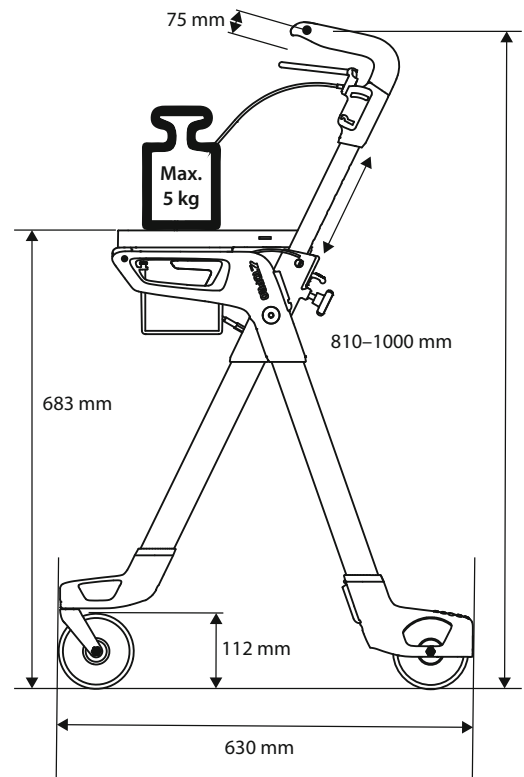
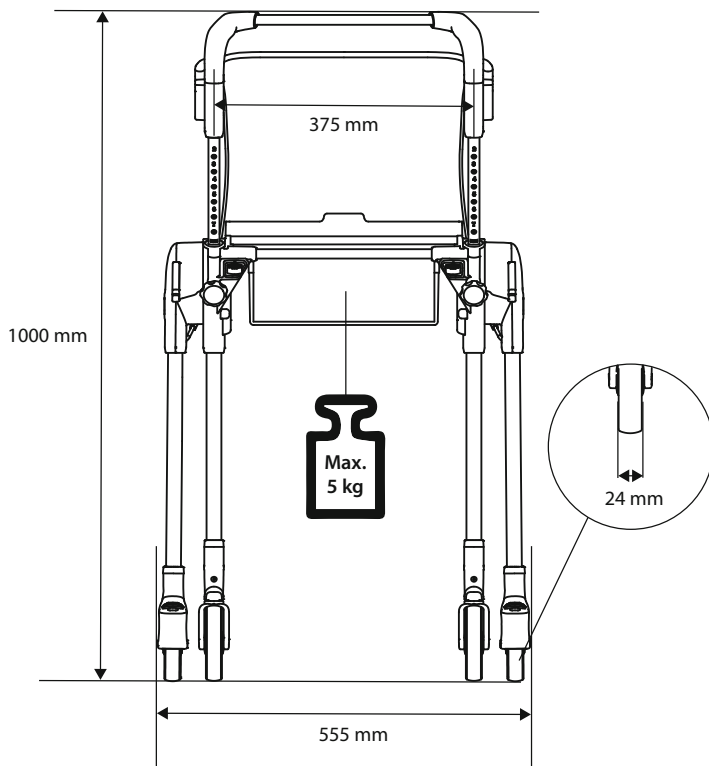
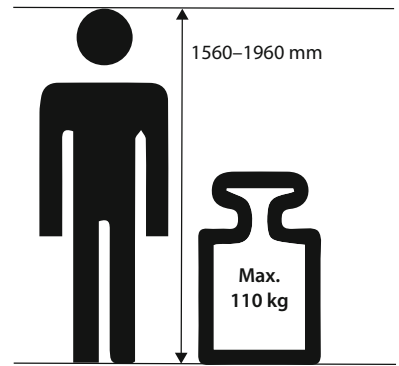
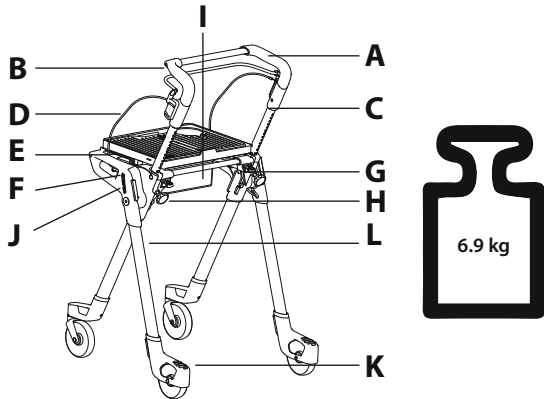
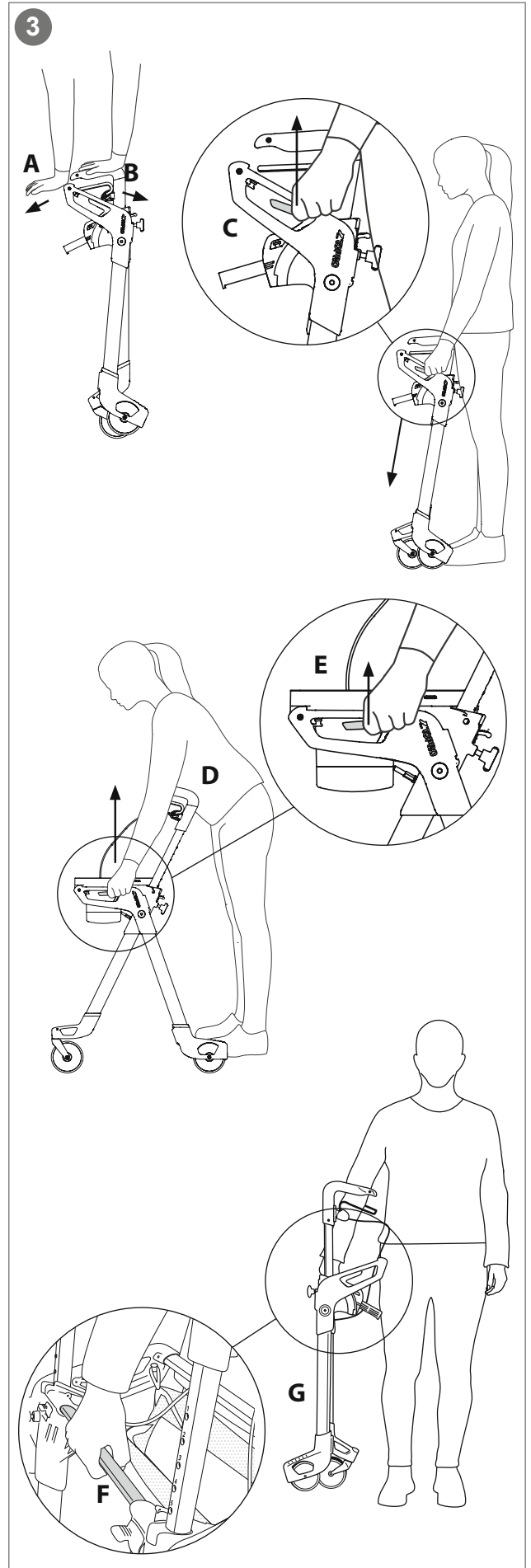
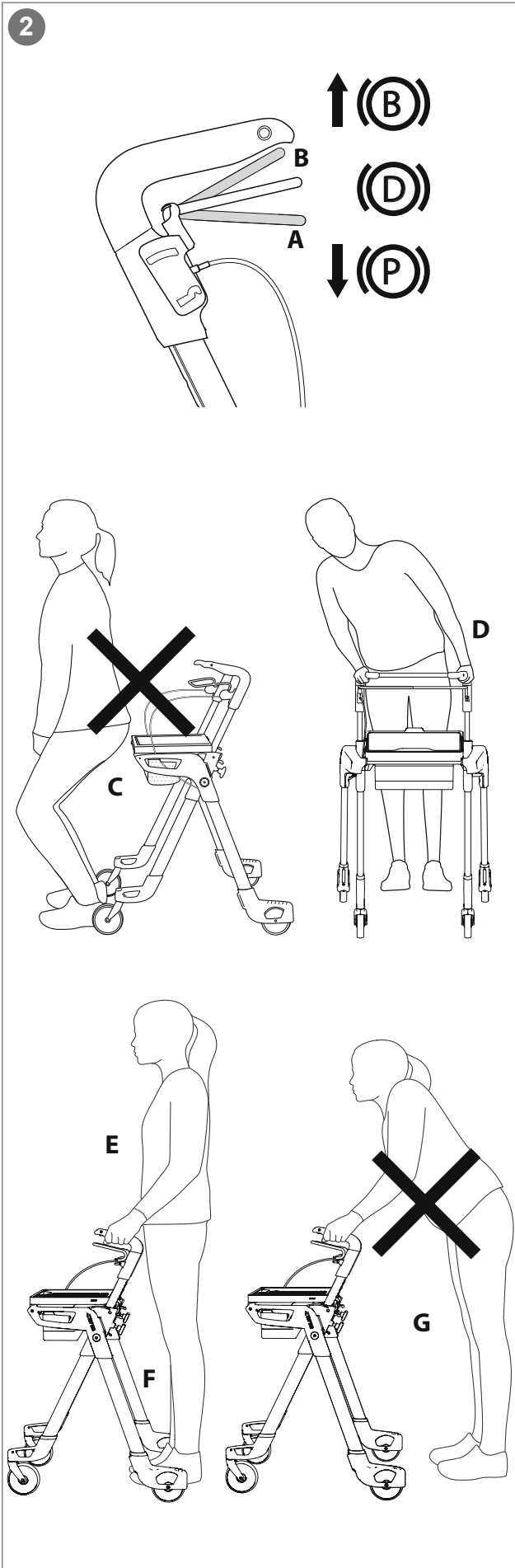
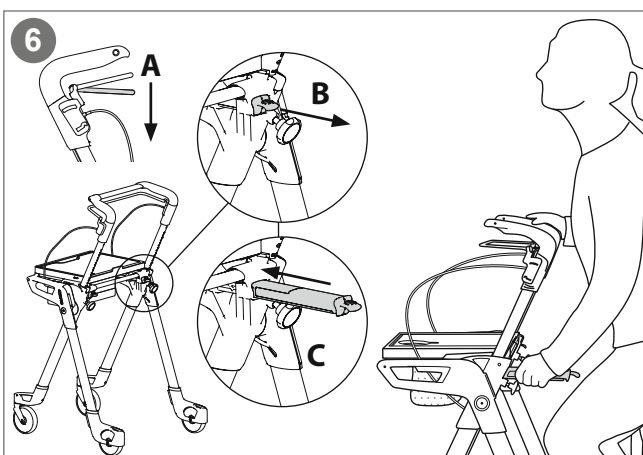
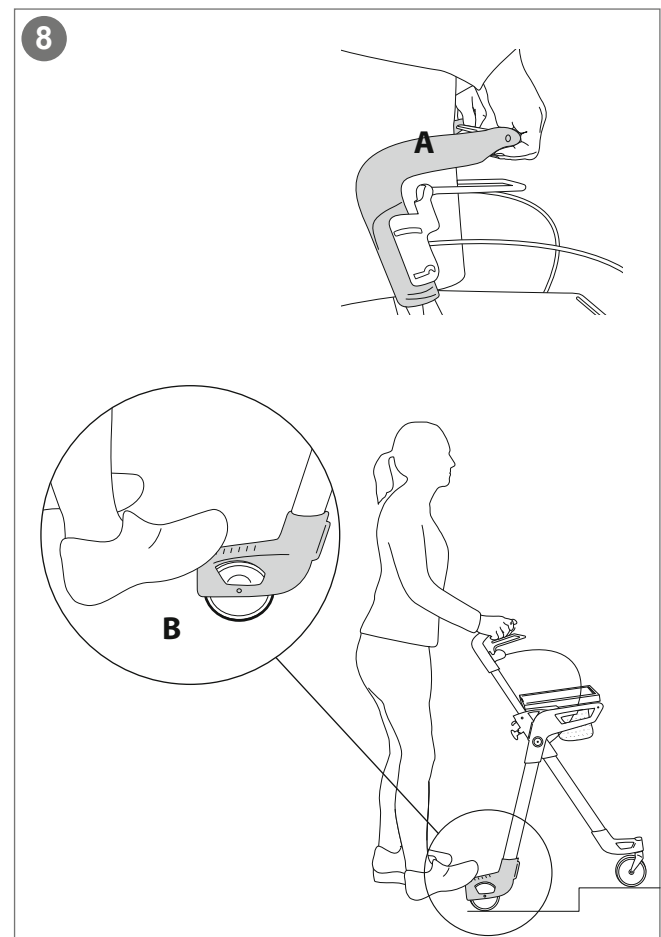
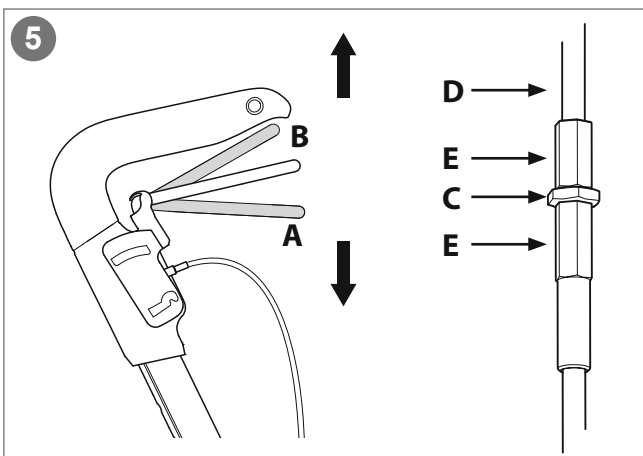
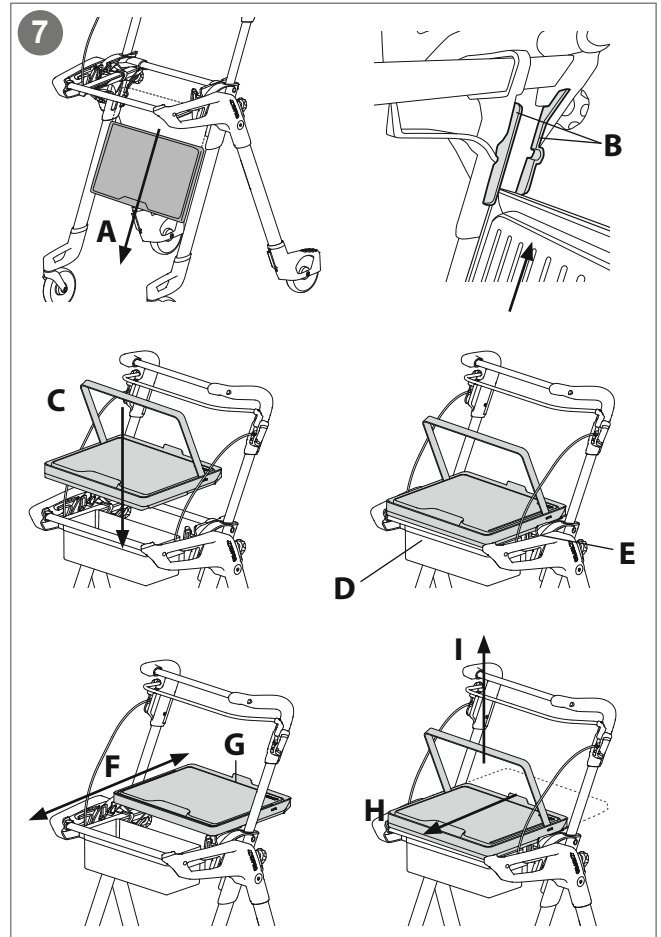
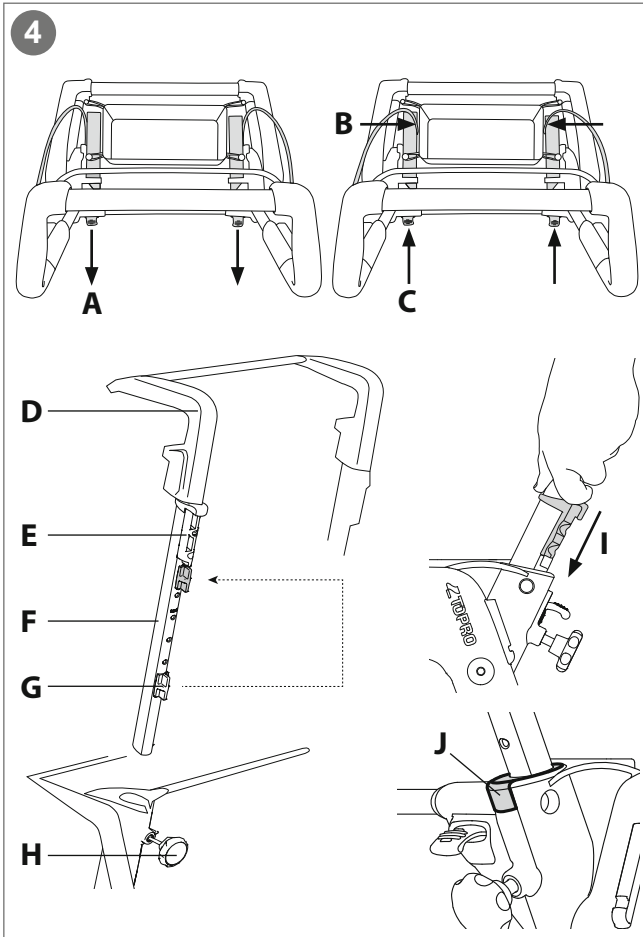
1

9

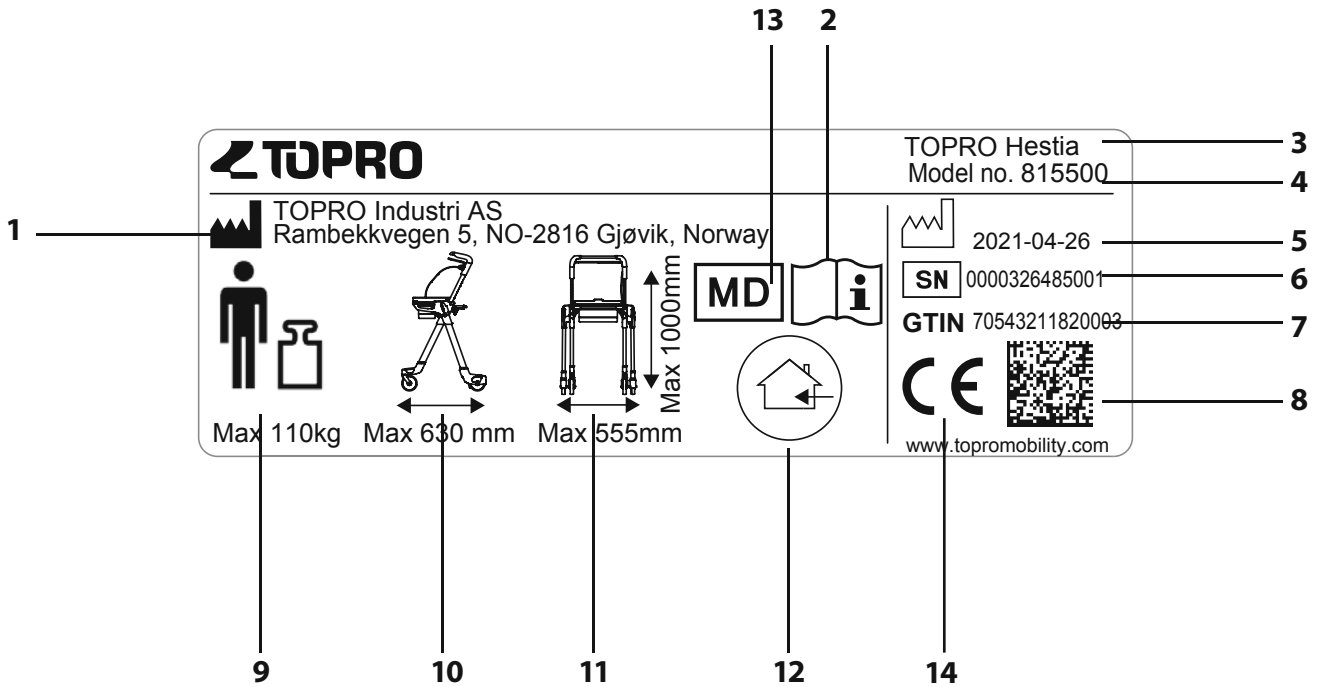Congratulations with your choice of a new TOPRO rollator

Every TOPRO product is developed, designed and manufactured in Norway. The product conforms to Medical Device Regulation (EU) 2017/745. It is tested and approved according to EN ISO 11199-2:2005. On receipt of the goods, please check your product. If you have any queries, please contact your mobility shop or dealer immediately. If you have problems reading this user manual, an electronic version is available on topromobility.com.

Intended use

The TOPRO HESTIA shall give support to users with reduced balance and/or reduced walking ability. The product is designed for indoor use. It is designed to be pushed, not pulled. The rise-up handles are intended to provide the user with supplementary support when rising up/sitting down. The tray provides an opportunity for the user to move items weighing up to 5 kg. The basket is designed for carrying items weighing up to a total of 5 kg. The intended user is an adult. Height and weight restrictions apply.

Contraindications: The product is not suitable for persons with low strength in arms, with very poor balance, or with considerable cognitive disabilities.

To get to know the rollator and learn how to use it properly, training by a professional is recommended.

0 The grey numbered circles in this user manual refers to the numbered information graphic and illustrations inside the cover

1 Get to know your TOPRO product

- | | |
|-------------------------|---------------------------------|
| A Walking handle | G Rise-up handles |
| B Brake lever | H Height adjustment knob |
| C Handle tube | I Storage net |
| D Brake cable | J Tray base |
| E Slidable tray | K Tilting function |
| F Folding lever | L Product label |

2 Safety

When folding/unfolding the rollator, be aware so you do not hurt your fingers.

Be aware not to put your fingers in open holes or between stationary parts.

Make sure that the height adjustment knob is tightened properly.

Ensure that the tray is placed correctly in its cradle before walking. The tray is not a seat.

When stationary, parked, and while using the rise-up handles always use the parking brake (A).

Do not use the rollator on stairs. Be careful when descending thresholds etc.

Check the brakes before use every time, and that the rollator locks in unfolded position.

Do not use the rollator to transport heavy load or persons.

Be aware that some parts of the rollator might feel hot or cold when exposed to extreme temperatures.

Do not modify the product as this might put your safety at risk and the guarantee will become void.

The experienced stability of the product will be lower when walking on a tilted surface.

Do not attempt to sit on the rollator (C).

To reduce the risk of falling, do not tilt the rollator too much sideways (D). For the same reason, bags etc. must never be hung on the handles.

To give the right support and safety, walk straight and make sure the rollator is kept as close to your body as possible (E). Walk between the rear wheels (F), not behind them (G).

Be aware that the rollator may roll faster than you when walking downhill, which may put you in risk of falling. Activate the driving brake to adjust the speed (B).

Be careful when wearing fluttering skirts and trousers as they could get caught in the wheels, which could lead to falling. Wear sturdy footwear to avoid injuries.

Do not use the rollator on slippery surfaces.

Keep the rollator away from fire and hot items.

Do not climb on the rollator.

3 Unfolding / Folding

Unfolding first time: The product is delivered in a transport position. In order to unfold the product place one hand on the crossbar between the walking handles (B) and the other hand on the crossbar at the front of the tray-base (A). Push the crossbars apart to initiate the unfolding. The product clicks into the normal folded position.

Unfolding: Place the product on its wheels, on level ground, and place yourself on the walking side of the product. Pull the two folding levers (C) positioned in the side openings of the tray-base. Press the tray-base down while squeezing the folding levers. The rollator unfolds and locks when the folding levers are released. To position the tray, see dedicated paragraph *Tray / Basket*.

Folding: Remove the tray. Place yourself, seated on a chair if deemed necessary, on the walking side of the product. While holding back the walking handle with your upper body (D), grip both folding levers (E) and pull the tray-base towards you. Release the folding levers and gently push the tray-base back down to verify that the folding levers are engaged, and the product cannot unfold. Apply the parking brake to ensure that the product does not roll away.

Carrying the rollator: The rollator must not be carried by the brake cables. Fold the rollator as described in section *Unfolding / Folding*. Carry the rollator by one of the crossbars (F) or frame sides (G).

4 Handles / Memory function

Height adjusting: Untighten both knobs (H) (approx. 2 turns). Adjust the walking handles (D) to a suitable height then pull them slightly up and down until you hear a clicking sound. Adjust both handles to the same height by comparing the number scale on each side. Tighten the knobs (H) again. *It is recommended that the handles are set to the same height as your wrists when your arms are hanging alongside your body.*

Memory Function: Decide on the height of the handles (see number scale). Count visible holes at the back of the handle tubes (F). Remove the tray. Extend the rise-up handles (A) and move the brake cables inwards (B). Hold the brake cables against the bag (B) while retracting the rise-up handles (C). Unscrew the height adjustment knobs (H). Pull the tubes (F) out of the frame. The fastening pieces (E) will now follow the handle tubes.

Move the fastening pieces (E) so that the number of holes above them equals to the previously counted holes. Fit the red memory chips (G) on the handle tubes in the first available hole under the fastening pieces (E). Push the handle tubes and fastening pieces (I) down and into the side frames (J)). Screw the height adjustment knobs (H) halfway in. Pull the handles (D) up until they stop and then tighten the knobs (H) properly. From now on, when the handles are in the lowest position, they will stop in the pre-set position when pulled up. Reposition the brake cables to the outside of the rise-up handles by extending/retracting the handles yet again.

5 Brakes

Make sure the brakes are working before each trip. The brake lever works on both rear wheels. Do not drive or push the rollator while the parking brakes are in use.

Parking brakes: Push the brake lever (A) down until the brakes lock.

Both rear wheels are now locked. Squeeze the brake lever upwards to release the parking brakes.

Driving brakes: Pull the brake lever (B) upwards to apply the brakes. Do not use continuously while the rollator is in motion.

Adjusting / checking: Adjusting the brakes is easiest done by turning the rollator upside down. Loosen the nut (C) on the brake cable. Tighten the brake cable (D) with the adjustment screw (E). Adjust both brake blocks evenly so the distance to the wheel surface is approximately 1.5 mm and then tighten the nut (C). Make sure the brakes are not adjusted too tight and pressing hard into the rubber of the wheels when the parking brake is on.

The wheels can be replaced if they are worn or damaged.

6 Rise-up handles

Always apply the parking brake (A) before operating the rise-up handles.

Extending: Extend the rise-up handles by pulling in the tab (B) situated next to the height adjustment knob. When the handle is fully extended it will be locked from slipping back in when pushed downwards.

In use: When the rollator is placed on levelled ground, the handle can act as a supplemental support when rising or sitting down.

Retracting: Retract the handle by lightly lifting and pushing the handle back in, until the handle clicks into place (C).

7 Tray / Basket

Tray transport position: Removing the tray is best done while seated. Apply the brakes and hold on firmly to the rollator. Push the tray firmly towards the floor (A). Placing the tray in the transport position is done by holding the tray with the tab (G) pointing forward and upward and aligning the tray sides with the ribs on the inside of the frame (B). Pull firmly up until the tray clicks into place on both sides.

Mounting: Lift the tray into position using the lifting handle (C). The tab (G) on the tray should face the walking side of the product. The tray is mounted by aligning tray sides with tray-base edge (E), the front of the tray should align with the front crossbar (D) and the arches in the corners.

Sliding: When placed on the tray-base the tray can be slid towards the user (F) using the integrated tab (G). The tray can be machine washed at up to 75 °C.

Removal: Push the tray fully forward (H). Lift the tray using the integrated lifting handle (I).

Basket: The basket can be removed, and machine washed gently at up to 60 °C. The basket is easiest to remove when the product is folded.

8 Lifting lip / Tilt function

Lifting lip: The handle has a geometry (A) that facilitates lifting the rollator across obstacles. Place your fingers around the lip and lift vertically.

Tilt function: Place the front of the foot on the tilt pedal (B) and pull the walking handles towards you. This helps to tilt the rollator over thresholds, edges etc. Please note: Take care when descending thresholds etc. Do not use the rollator on stairs.

Transport

The rollator should be transported in the folded and locked position (see paragraph 3 – Unfolding / Folding). Be careful when loading the rollator into a vehicle, giving attention to securing it.

Storage

The rollator should be stored in an upright position. If stored outdoors it must be under a cover to protect it. Do not leave your rollator unattended. Do not put heavy items on top of the rollator during storage.

Cleaning

The rollator can be hosed and washed with normal household detergents. Do not use abrasives. Avoid flushing directly against bearings (wheels, front forks).

Disinfection

Disinfection must only be carried out by authorized personnel and by wearing sufficient protective equipment. The surface of the rollator must be cleaned with a disinfectant containing 70-80% ethanol. We advise against disinfectant that contains chlorine and phenol.

The manufacturer cannot be held responsible for any damage or injuries that might occur using harmful disinfectant or disinfection carried out by unauthorized personnel.

Inspection / Maintenance / Re-use

It is recommended that inspection and maintenance is carried out regularly (frequency is determined by how and how often the rollator is used). Check the following items: Frame, screws, handles, handle tubes, brakes, brake parts, wheels and accessories. This also applies when the rollator is to be prepared for re-use. Please see detailed instructions in this user manual about maintenance of brakes and cleaning/disinfection. The recommended maintenance is not a requirement, and no preventive maintenance is required provided that the rollator is used as intended according to this user manual.

Material / Recycling

The rollator is made of anodized aluminium profiles, parts of PA6+30GF, PP, TPE and Polyester. Most parts can be recycled. Dispose the rollator and its packaging according to the applicable regulations in your country. For information, please contact your public authorities.

Warranty

TOPRO Hestia is guaranteed free of faults and defects for 7 years. Damage caused by incorrect use or parts that are exposed to natural wear and tear (e.g., brake block, brake cable, wheel, handles, storage net and tray) are exempt from the 7-year warranty. For repairs during the warranty period, please contact your local mobility shop or dealer. The warranty is void if unauthorised spare parts and accessories have been or are being used on the product. The expected life span of the product is estimated to be 10 years if it is used in the correct manner according to this user manual and its safety and maintenance instructions.

9 Product label

- 1 - Manufacturer
- 2 - Consult Instructions for use
- 3 - Model name
- 4 - Model number
- 5 - Date of manufacture
- 6 - Serial number
- 7 - Global Trade Item Number
- 8 - GS1 DataMatrix
- 9 - Maximum user weight
- 10 - Maximum length of rollator
- 11 - Maximum width / height of rollator
- 12 - Intended for indoor use
- 13 - Medical Device
- 14 - CE labelling

Accessories

	TOPRO Art. No.
Crutch holder	815358
Anti-slip mat for tray	815384
Bottle holder	815843
Name tag	814024

Accessories mounted on the rollator may influence stability, we therefore advise you to use them with care. Accessories can be ordered separately to enable a TOPRO HESTIA to be customised to individual needs. Contact your mobility shop, dealer or TOPRO for an updated overview of available accessories or visit our homepage topromobility.com.

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TOPRO Limited
TOPRO Mobility AB